EN

Intended use

DentalBiome GelPro is used in support of caries and periodontitis treatments. It is also indicated for the prevention of caries and periodontal disease.

Its gel based formula contains *L. paracasei* GMNL-33 (ADP-1), which inhibits pathogenic bacteria associated with dental caries and periodontal disease.

Presentation

Outer box containing 5 vials. Each vial contains a separated solid phase and liquid phase.

Composition

Solid phase: *Lactobacillus paracasei* GMNL-33 (ADP1, 10 billion cells per vial), guar gum, maltodextrin & magnesium stearate.

Liquid phase: Purified water, natural mint flavour, potassium sorbate, sodium benzoate.

Instructions for use

Please note that the 3 ml syringe and the tip are not included. You will need to purchase them separately. We recommend to use the BD Plastipak 3 ml Luer Lok syringes (REF 309658) and the Miraject Endo Luer tips (REF 254215), or any equivalent system.

- 1 Take a 3 ml syringe and assemble it with an appropriate tip beforehand
- 2 Screw down the cap so that the powder falls down in the liquid phase
- 3 Shake vigorously for 5 seconds
- 4 Aspirate the resulting mixture with the syringe equipped with the tip
- 5 Wait for at least 3 min
- 6 Apply the gel in the gingival pockets & wait for at least 5 min

See figures below

Dosage

1 vial per application

Contraindications

DentalBiome GelPRO must not be used in severely immunocompromised or immunosuppressed patients as well as in patients with known hypersensitivity to any of the ingredients of the product.

Precautions

- For professional use only
- This class 1 medical device is not a pharmacological treatment. If such treatment is required, this medical device can be administered concomitantly
- Although no side effects are known, it is recommended to not use the product in children under 6 years old, or during pregnancy and breastfeeding, unless otherwise indicated by a healthcare professional
- Do not use the product if the vials are opened or damaged
- Keep product out of reach of children

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